IT

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EN

PREVENTION AND TREATMENT OF
HYPERTROPHIC SCARS

SKIN REPAIR

Following a wound of any sort, a healing process begins which concludes with scar formation. The healing phase takes anywhere from 21 days to two years.

HYPERTROPHIC SCARS

Hypertrophic (and keloid) scars are the result of excessive collagen production invading the intact skin around the injured area. Scar formation takes place during the healing phase.

INDICATIONS

Resolve Gel Siliconico allows for correct management of the healing phase. **Resolve Gel Siliconico** can be used on the scars produced by any type of wound (surgical, caesarean section, abrasions, scrapes, burns, animal bites, laceration, cuts, acne). It is recommended that treatment be continued for at least 2 months for the best aesthetic result.

WHAT IS RESOLVE GEL SILICONICO

Resolve Gel Siliconico is a medical device containing a transparent silicone gel. It uses a semi-occlusive, moisturizing mechanism. This physical mechanism keeps the temperature of the healing zone constant and prevents the loss of moisture from the tissues.

INGREDIENTS

Aqua, Cyclopentasiloxane, Dimethicone, Sodium Polyacrylate, Trideceth-6, PEG/PPG 18/18 Dimethicone, Phenoxyethanol, Ethylhexylglycerin, Tocopheryl Acetate.

INSTRUCTIONS FOR USE

1. Clean and dry the scar well.
2. Remove the protective cap of the tube.
3. Squeeze the tube.
4. Apply a small quantity of gel to the scar evenly and rub to obtain a thin layer.
5. When the gel is dry, it can be covered with clothes.
6. Repeat treatment 3-4 times a day.

⚠ WARNINGS

Resolve Gel Siliconico must be applied only on wounds that have closed and are in the process of

GEL SILICONICO

healing. Do not apply **Resolve Gel Siliconico** to mucose membranes, to the area around the eyes, or to skin treated with medicinal products in the form of creams, emulsions or ointments. In the case of maceration, reddening or rash, discontinue treatment until the symptoms have disappeared. **Resolve Gel Siliconico** is contraindicated in subjects with dermatological disorders incompatible with long-term use of the product. Keep out of reach of children.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Reference Member State.

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